

BODY SHAPE AND TIGHT



MC1 Multipower is equipped with the innovative BODY SHAPE AND TIGHT handpiece, the state of the art tool for body shaping and tightening.

Body Shape and Tight handpiece is made by a disc on which are placed 3 circular ultrasonic transducers with sequential activation, all behaving as separate US sources.


The sequential activation allows to exploit the highest peak power pulses, emitted in turn by each transducer, thus inducing the effect of a tapping massage on the treated area. The results of the treatment with MC1 Multipower Body Shape and Tight handpiece are the improvement of the epidermal texture and the re-organization of the underlying dermis, a better skin tone due to the induction of vasoactive processes, an increase of collagen fibers and a higher fibroblast stimulation. Body Shape and Tight handpiece supplies also radiofrequency, which enforces and improves the action of the ultrasound treatment. In fact, the tripolar electromagnetic field applied to the tissue generates heat on the treated area by increasing its internal temperature. This reaction causes a denaturation of collagen fibers with their subsequent re-organization: they actually become shorter and thicker as a result of a noticeable tightening effect.

Radiofrequency also acts on the tone of the skin, improving its elasticity and smoothness, on cellulite and skin laxity.

HOOP



REG. N. 1889
UNI EN ISO 9001: 2008
UNI EN ISO 13485: 2012

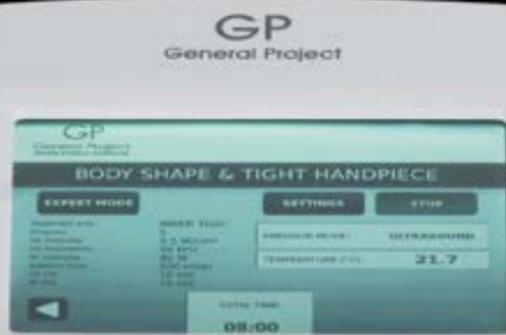

GP

General Project
ADVANCED BIOMEDICAL TECHNOLOGIES

General Project s.r.l.

Via della Gora 15/19 - 50025 Montespertoli (Firenze) Italy
Tel. +39 0571 675076 - Fax +39 0571 675077
www.generalproject.com - info@generalproject.com

GP

General Project
ADVANCED BIOMEDICAL TECHNOLOGIES

ADVANCED SHAPING, TIGHTENING AND LIFTING TECHNOLOGY

MC1MULTIPOWER

82022372UK

Ultrasound and Radiofrequency
Body and Face Platform

MC1MULTIPOWER


The efficient, safe and easy to use US and RF platform for non-surgical body shaping, skin tightening and face lifting.




MC1 Multipower is an innovative and efficient platform which combines two different technologies: Ultrasound and Radiofrequency.

Equipped with 4 different handpieces, each one carefully studied and tested to perform its specific action, MC1 Multipower treats the whole body, from head to toe, reshaping, tightening and lifting.

ultrasound action

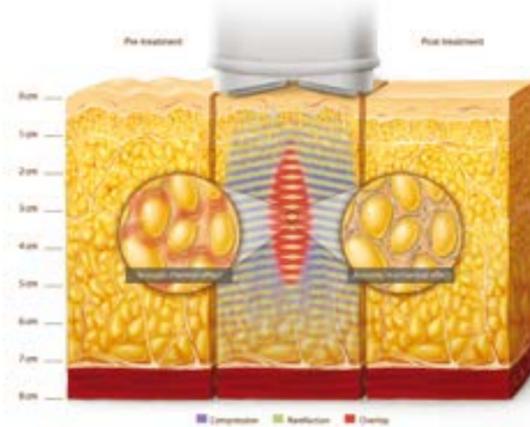

radiofrequency action

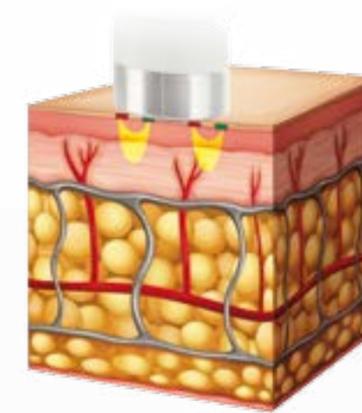

Exploiting both US and RF technologies, MC1 Multipower performs non-invasive face and body treatments focusing on:

- LOCALIZED FAT AND CELLULITE**
- CIRCUMFERENCE REDUCTION**
- SUBCUTANEOUS MICROCIRCULATION**
- THIN WRINKLES**
- FACE AND NECK LAXITY**
- LYMPHATIC DRAINAGE**

The specific US emission, particularly studied for MC1 Multipower, causes a significant shrinking of subcutaneous adipocytes, thereby activating metabolic processes which lead to the natural elimination of the fat waste, thus fighting against localized adiposity and cellulite, ultimately resulting in a body re-shaping effect.

On the other hand, the combined action of US and RF, peculiar feature of MC1 Multipower, allows in-depth treatments which can effectively reduce wrinkles and face and neck skin laxity. Clinical and histological studies showed that the synergic action of US and RF can compact dermis connective fibres, specifically collagen and elastin, thus granting the skin tightening effect.



4 HANDPIECES

FOR FACE AND BODY SHAPING,
TIGHTENING AND LIFTING



BODY SHAPE AND TIGHT

Working with both US and RF, it is equipped with 3 low frequency US transducers; the unique features of this handpiece are the radial configuration of the 3 transducers and their sequential activation which allows them to focus on a specific body area with a tightening effect.



BODY SHAPE

Thanks to the joint action of 2 low frequency US and to its peculiar shape, it focuses on the treated area to act efficiently against localized fat.



FACE LIFTING

Exploiting both the performance of US and RF, it carries out in-depth treatments to significantly reduce thin wrinkles and face and neck laxity.



DRAINAGE

Provided with a devoted biocompatible membrane, it performs the draining massage, both on face and body, and the opening of the lymphatic ganglia, allowing the disposal of fat waste after the US treatment.